## REVISIONES DE LENTES DE CONTACTO

---

### Revisiones

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---

### Follow-up

Prior to a follow-up examination, the contact lenses should be worn for at least four continuous hours and the patient should be asked to identify any problems which might be occurring related to contact lens wear.

With lenses in place on the eyes, evaluate fitting performance. Examine the lenses closely for surface deposition and/or damage.

After the lens removal, instill sodium fluorescein into the eyes and conduct a thorough biomicroscopy examination.

The presence of vertical corneal striae, corneal staining and/or limbal-conjunctival hyperemia can be indicative a poorly fitting lens. The patient should be re-fitted with a more appropriate lens.

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## RECOMMENDED INITIAL WEARING SCHEDULE

### Schedule

The wearing schedules should be determined by the eye care practitioner.

The following sequence is recommended as a guideline.

During Waking Hours

Day	Hours
1	4-8
2	6-10
3	8-14
4	10-15
5	12-all waking hours
6	after - all waking hours

Lenses should be removed daily for cleaning and disinfecting (according to lens care system instructions) before wearing.

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## HOW ARE CONTACT LENSES-SUPPLIED ?

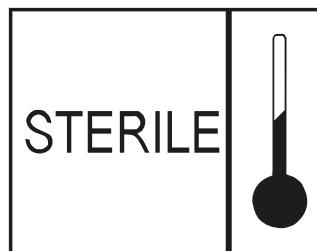
### Deliveries

Each Soft Contact Lens is supplied sterile, using steam heat, immersed in buffered saline solution (0.9%).

Each RGP Contact Lens is supplied non-sterile immersed in Multi-Purpose Solution.

If the patient is sensitive to any ingredient in the solution, the lens should be removed from the plastic container upon receipt, rinsed with fresh saline solution, cleaned with a cleaner and placed in another prescribed disinfecting solution prior to dispensing. Follow the manufacturer's instructions on the disinfecting solution label.

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## WARNING OVER TIGHT (STEEP) (NON-MOVING) LENS

---

### Light Tight Fitting

For different reasons, a contact lens during the course of the day may begin to fit more tightly onto the surface of the eye. This may be because the lens was too tight fitted to begin with, or it may be related to increasing dryness of the lens and eye as the day goes by. If the lens reaches a point where it stops moving on the eye, several complications may happen:

- redness
- eye irritation
- burning
- dry sensation

Then...

INCREASE BASE CURVE OR REDUCE DIAMETER

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## CLINICAL ASSESSMENT

---

\* Vision should be crisp and clear after the blink.

\* The eye should be white ALWAYS, after wearing contact lenses 8 or 10 hours .

Initial or temporary discomfort may be caused by a foreign body under the lens surface. The lens should be removed, rinsed and reinserted. If the discomfort persists, the patient should consult the eye care professional before returning to lens wear.

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## REPORTING OF ADVERSE REACTIONS

### Quality Management

All serious adverse experiences and adverse reactions observed in patients wearing Servilens Contact Lenses should be reported to:

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EC Medical Device Regulation  
Standards, Directives and  
Technical Guides