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**INSTRUCTION FOR USE 47841**  
Contact lenses 1 Day

Information leaflet, to be provided to wearers of contact lenses by their contact lens specialist/optometrist

**PLEASE READ CAREFULLY AND KEEP THIS INFORMATION FOR FUTURE USE. FOLLOW THE INSTRUCTIONS OF YOUR EYE CARE PROFESSIONAL. STRICTLY. DAILY DISPOSABLE LENSES ARE INTENDED TO BE WORN ONLY ONCE AND THEN DISCARDED. DAILY DISPOSABLE LENSES ARE NOT DESIGNED TO BE DISINFECTED AND WORN MORE THAN ONE TIME.** Daily Disposable lenses worn more than one time may become damaged and could put you a risk of contact lens care and handling related eye infection. Contact lenses should always be fitted by an eye care professional, who will recommend the wearing and replacement schedule best suited for your needs. These lenses are intended for vision correction and can be used for cosmetic effects in people with healthy eyes and always under the supervision of an eye care professional. **INSTRUCTIONS FOR HANDLING YOUR CONTACT LENSES.** See in figures 1 – 5. Do not use contact lenses if pack is opened or damaged. Do not use the contact lens if the lens is damage. Your eye care professional should provide you with detailed contact lenses insertion and removal instructions, as well as information on the safe use of your lenses. Wear the contact lenses during the time indicated by your eye care professional. Check your vision periodically with your eye care professional and follow the instructions use. **Blister:** gently shake the blister pack before opening. Peel back the foil lid. Pour the lens into the palm of your hand or if necessary, gently slide the lens out of the container, or open as directed by your eye care professional. **CONTACT LENSES MAINTENANCE** • **NEVER** allow contact lenses to come in contact with non-sterile water (including tap water) as microbial contamination can occur, which may lead to permanent eye damage•. Ask your eye care professional about wearing contact lenses during water sporting activities. Remove lenses when exposed to noxious or irritating vapour. **IMPORTANT PRECAUTIONS** When you are wearing contact lenses, check your eyes every day. Your eyes should look well, you should feel comfortable with them and your vision should be clear. If it is not, remove your lenses immediately and contact your eye care professional. Unusual symptoms, if ignored, can lead to serious complications. Do not use contact lenses in: corneal dystrophies, corneal ulcers, inflammation of cornea or the eye, allergic eye conditions. Do not share your contact lens with anyone else. Do not wear your lenses when using eye drops (unless advised to do so by a medical practitioner). Do not use saliva to wet your lenses. **Daily USE. FOR ONLY USE DURING THE DAY** (Do not use contact lenses while you are sleeping). No special conditions are required for its disposal.

**Product name:** see packing label. **Material name:** see packing label Administration: suitable for eye use. **Wearing schedule:** the lenses are suitable for daily wear (cannot be worn overnight). Follow the wearing schedule provided by your contact lens specialist/optometrist. Indication: to correct myopia refractive disorders and optimise vision ability. Sterility: lenses are sterilised by steam (sterility is only guaranteed in the closed and undamaged sealed package). Do not use the lenses if the packaging has been tampered with. **Replacement:** according to instructions from the contact lens practitioner. Lenses identified on the carton box as “for daily replacement” are indicated for one day use only.