

Lavar manos. Wash your hands. n-2 Adaptación/Fitting

n-3 Extracción/Lens removal

n-3' Frotando/rubbing

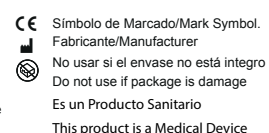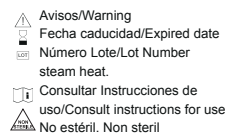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RECOMMENDED REPLACEMENT FOR CONTACT LENSES AND MATERIALS ISO 18369-1. Patient Instructions 47840 IFU RGP GB
 ST K 70, ST K 70 Toric, ST K 70 Multifocal (365 days) (Focon III 3). ST K 100, ST K 100 Toric, ST K 100 Multifocal (365 days) (RofluFocon D), Lens 180 GP + S (Filcon I 1 + Focon 4 180 days) ST K 40 (365 days Focon III 3). Contact lenses in an ASEPTIC SOLUTION. Clean and rinse the lenses before use. Contains: EDTA 0.10%, PolyHexamide 0.0002%.

STRICTLY FOLLOW THE INSTRUCTIONS OF YOUR EYE CARE PROFESSIONAL. PLEASE READ CAREFULLY AND KEEP THIS INFORMATION FOR FUTURE USE. Contact lenses should always be fitted by an eye care professional, who will recommend the wearing and replacement schedule best suited for your needs. These lenses are intended for vision correction, always in people with healthy eyes and under the supervision of an eye care professional. INSTRUCTIONS FOR HANDLING CONTACT LENSES. See Figures 1 – 3. Do not use the contact lens if the pack is opened or damaged. Do not use the contact lens if the lens is damaged. If lenses dry out, rewet them with a disinfecting and hydrating solution. Ask your eye care professional. Your eye care professional should provide you with detailed contact lens insertion and removal instructions, as well as with information on the safe use of your lenses. Blister Gently shake the blister pack before opening. Peel back the foil lid. Pour the lens into the palm of your hand or, if necessary, gently slide the lens out of the container. CONTACT LENSES MAINTENANCE • A CORRECT CARE OF YOUR CONTACT LENSES IS ESSENTIAL • Your eye care professional should choose the lens care system most suitable for you (if applicable). Please always follow the labelling instructions on your lens care products. • Do not use heat disinfection. • NEVER allow contact lenses to come in contact with non-sterile water (including tap water) as microbial contamination, which may lead to permanent eye damage, can occur. • Do not change the contact lens maintenance system without consulting your eye care professional. • Ask your eye care professional about wearing contact lenses for sport activities. • Remove your lenses when exposed to irritating vapour. IMPORTANT PRECAUTIONS When you are wearing contact lenses, check your eyes every day. Your eyes should look well, you should feel comfortable, and your vision should be clear. If it is not, remove your lenses immediately and contact your eye care professional. Unusual symptoms, if ignored, can lead to serious complications. Do not use contact lenses in corneal dystrophies, corneal ulcers, corneal or eye inflammation, and eye allergy conditions. Do not share your contact lens with anyone else. Do not wear your lenses when using eye drops (unless advised to do so by a medical practitioner). Do not use saliva to wet your lenses. DAILY USE. USE CONTACT LENSES ONLY DURING THE DAY (Do not use contact lenses while you are sleeping).

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